



ASPEN VISTA® MULTIPOST COLLAR

MULTIPOST CERVICAL COLLAR
MULTIPOST ZERVIKALORTHESE
CUELLO CERVICAL MULTIPOST
COLLARE CERVICALE MULTI-POSIZIONE
COLLIER CERVICAL À PLUSIEURS POSTES
MULTIPOST NEKBRACE
VÍCE-POLOHOVÝ KRČNÍ LÍMEC
MULTIPOST HALSHVIRVELKRAVE
ПОЛУКОМБИКО ЛАШЕНИКО КОЛАРО
MULTIPOST CERVICAL KRAGE








KOLNIERZ ORTOPEDYCZNY WIELOPODPOROWY
MULTIPOST KAULATUKI
MULTIPOST NACKKRAGE
МНОГОПОЗИЦИОННЫЙ ШЕЙНЫЙ КОРСЕТ
COLAR CERVICAL MULTIPOST
多重支撐頸托

ASPHEN VISTA CERVICAL COLLAR FRONT
ASPHEN VISTA MULTIPOST BACK PANEL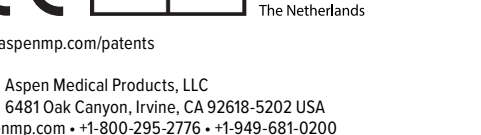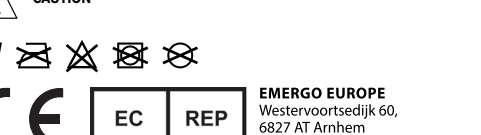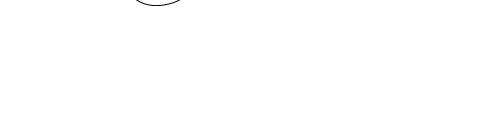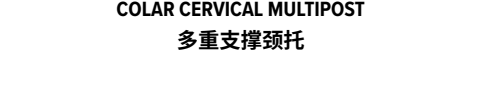

EMERGO EUROPE
Westervoortsepoel 60,
6827 AT Arnhem
The Netherlands

1

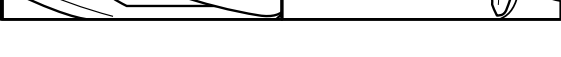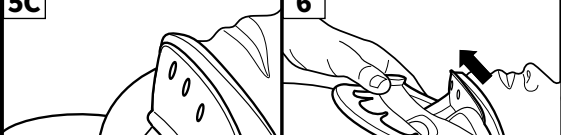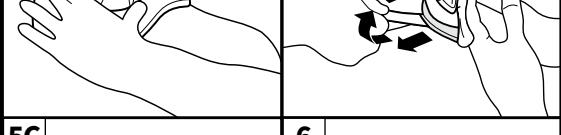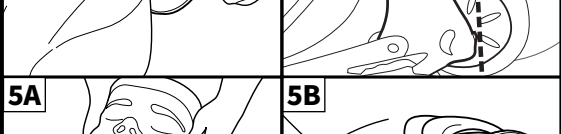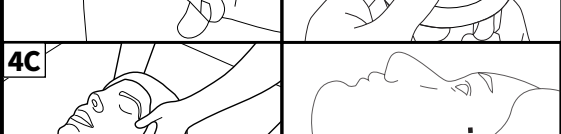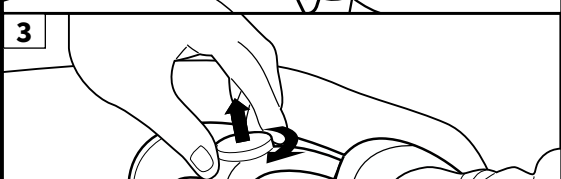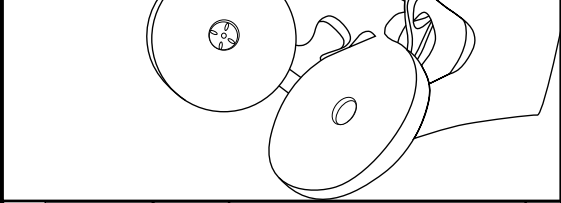ENGLISH

1. PREPARATION
Pre-form the back panel so plastic side pieces are slightly curved.
NOTE: Ensure head is in neutral alignment throughout the entire application of this product.

2. PLACEMENT
The front panel should be at the lowest setting to begin. Slide the collar up the chest and pivot up and under the chin. Position the sides up and over the trapezius muscles. It is OK if the chin piece is not touching the chin at this step. **It is not necessary to bend the sides of the collar outward or inward during application.**

3. SIZING
While holding the collar against the chest, pull out the dial to unlock. Turn the dial clockwise to raise chin piece. When the chin piece is supporting the chin, release the dial and it will self-lock.

4. MULTIPOST BACK PANEL
NOTE: Do not adjust on the patient. If height adjustment is necessary - remove from patient, adjust and reapply.
A. Push down on lock release and slide yellow button left to unlock.
B. Adjust height up or down if needed for best fit. To lock, slide yellow button right.
C. With the head in neutral alignment, press the back panel down onto the bed and side it behind the patient's neck. Ensure back panel has good contact with trapezius.
FOR REFERENCE: Start by aligning top of the back panel with the middle of the ear.

5. TIGHTENING
A. While holding the front panel in place, attach the loop strap on each side.
B. To tighten, anchor your fingers in on the collar and peel back the loop strap on one side, tighten and reattach. Tighten the other side equally and repeat as needed to ensure a snug, symmetrical fit.
C. Check to see that the back of the chin piece is not pressing inward on the throat. If it is, lift the back ends of the side panels up and off the trapezius muscles.

6. TIPS
A. The patient's chin should be close to the front of the chin piece but the back of the chin piece should not press on the patient's throat.
B. All slack between the front and back panels must be removed to ensure a secure fit.
C. The patient often changes position, so it may be desirable to readjust the height and the back panel straps.

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Pat. aspemmp.com/patents

EMERGO EUROPE
Westervoortsepoel 60,
6827 AT Arnhem
The Netherlands

©2023 Aspen Medical Products, LLC. All Rights Reserved.
ACO113F revised 03/2023

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIMITATIONS ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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