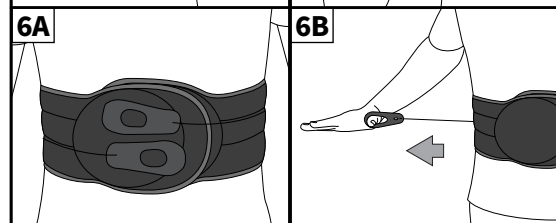
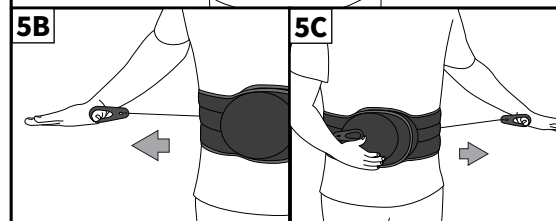
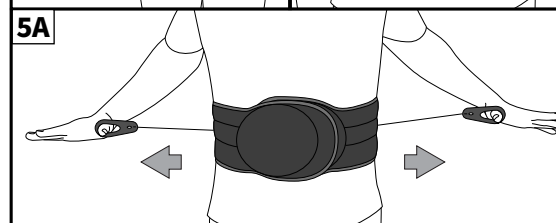
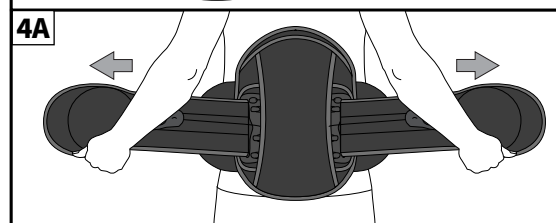
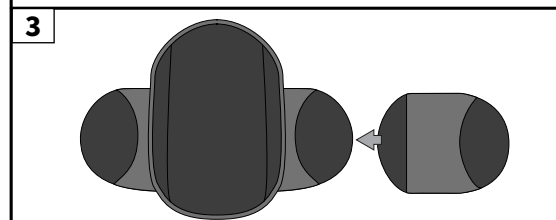
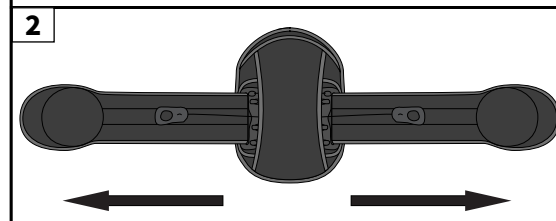
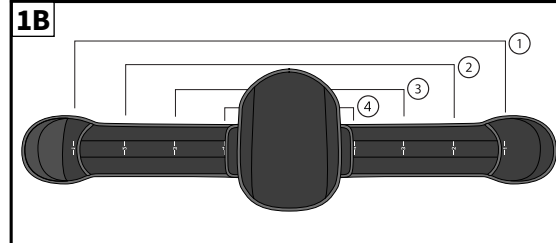
**ASPEN SIERRA™ 637 LSO  
ADJUSTABLE**

**LUMBAR SACRAL ORTHOSIS**  
**LUMBAL-SAKRALE ORTHESE**  
**ÓRTESIS LUMBOSACRA**  
**ORTÈSE LOMBO-SACRÉE**  
**LUMBALE SACRALE ORTHÈSE**  
**BEĐERNÍ KRÍŽOVÁ ORTÉZA**  
**LUMBAL SAKRAL ORTOSIS**  
**ΟΣΦΥΪΚΗ ΟΡΘΩΣΗ ΟΣΤΕΟΥ ΟΣΤΟΥ**  
**LUMBAL KORSRYGGÖRTROE**  
**ORTEZA ŁĘDŹWIOWO-KRZYŻOWA**  
**LANNERANGON SAKRAALINEN ORTOOSI**  
**LUMBAL SAKRAL ORTOS**  
**ОРТЕЗ ДЛЯ ПОЯСНИЧНО-КРЕСТЦОВОГО ОТДЕЛА ПОЗВОНОЧНИКА**  
**ÓRTÈSE SACRO LOMBAR**  
**腰骶矫形器**

**ENGLISH**  
**1. SIZE ADJUSTMENT**  
A. To adjust the size of the Aspen Sierra LSO, peel up the inside end of the side panel on each side. Adjust to achieve a proper fit and reattach.  
B. Size ranges are only guidelines. Adjust as needed to achieve a snug fit without too much overlap.

SIZING GUIDE			
Size	Circumference	Size	Circumference
1	28 – 37 in	71 – 94 cm	
2	36 – 43 in	91 – 109 cm	
3	42 – 48 in	107 – 122 cm	
4	47 – 54 in	119 – 137 cm	

**2. PREPARATION**  
Place the pull tabs 2 inches (5 cm) from the plastic in the back of the Aspen Sierra LSO brace. Pull the ends of the brace away from each ensuring that it is fully extended.  
**3. LATERAL PANEL SIZING**  
Add or remove panels for appropriate sizing.  
**4. APPLICATION**  
A. Apply the fully extended brace making sure the back panel is centered and low on the back. Apply the brace tightly around the abdomen, the right side overlapping the left.  
B. When applied properly, both the upper and lower edge of the brace should be snug. If there is any gap, reapply the brace and angle the ends to ensure a snug fit at both the upper and lower edge.  
NOTE: For certain patients, the ends must be steeply angled.  
**5. TIGHTENING**  
A. Slip your thumbs through the holes in the pull tabs. Pull away from your body.  
B. The left pull tab controls lower compression and the right pull tab controls the upper compression. Adjust the compression as needed.  
C. When tightened, secure the pull tabs to the loop in front of the brace.

**6. ADJUSTMENT**  
A. Adjust the upper and/or lower compression by loosening or tightening the pull tabs as needed.  
B. When patient is seated, slightly loosening the upper portion may be beneficial.  
**7. MODIFICATION OPTIONS**  
This product may be substantially modified by an individual with expertise. For more information, please visit [www.aspenmp.com/education/product-modifications](http://www.aspenmp.com/education/product-modifications).

**INTENDED USE**  
To provide motion restriction and stability of the lumbar-sacral spine.  
**INDICATIONS**  
Postoperative support, Deconditioned trunk musculature, Post laminectomy syndrome, Surgical Decompression procedures, Non-surgical Decompression procedures, Degenerative disc disease, Bulging or herniated disc, Non-specific low back pain, Chronic low back pain, Fracture management, Spondylolysis, Spondylolisthesis, Spinal stenosis, Ligament sprains / Muscle strains, Spondylosis (Osteoarthritis/Degenerative Joint Disease), Facet syndrome, SI dysfunction, Sacroiliitis, Radiculopathy, Injections, Radiofrequency Ablation, Lumbar Multifidus Muscle Atrophy

**CONTRAINDICATIONS**  
Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

**PRECAUTIONS**  
**READ INSTRUCTIONS BEFORE USE.** Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

**CARE INSTRUCTIONS**  
Hand wash only at 30° C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

**MATERIALS**  
Nylon, Polyester, HDPE, PVC, Polyurethane, UHMWPE.

**NOT MADE WITH NATURAL RUBBER LATEX.**

**DISPOSAL**  
Product may be safely disposed in accordance with local laws.

**REPORTING**  
Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

**LIMITED WARRANTY**  
Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product (part).

**Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:**  
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution.  
(b) The product is altered;  
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or  
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

**THE REMEDIATION OF THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT, ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIMITED TO THIS LIMITED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.**

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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**INDICATIONS**  
Postoperative support, Deconditioned trunk musculature, Post laminectomy syndrome, Surgical Decompression procedures, Non-surgical Decompression procedures, Degenerative disc disease, Bulging or herniated disc, Non-specific low back pain, Chronic low back pain, Fracture management, Spondylolysis, Spondylolisthesis, Spinal stenosis, Ligament sprains / Muscle strains, Spondylosis (Osteoarthritis/Degenerative Joint Disease), Facet syndrome, SI dysfunction, Sacroiliitis, Radiculopathy, Injections, Radiofrequency Ablation, Lumbar Multifidus Muscle Atrophy

**CONTRAINDICATIONS**  
Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

**PRECAUTIONS**  
**READ INSTRUCTIONS BEFORE USE.** Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

**CARE INSTRUCTIONS**  
Hand wash only at 30° C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

**MATERIALS**  
Nylon, Polyester, HDPE, PVC, Polyurethane, UHMWPE.

**NOT MADE WITH NATURAL RUBBER LATEX.**

**DISPOSAL**  
Product may be safely disposed in accordance with local laws.

**REPORTING**  
Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

**LIMITED WARRANTY**  
Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product (part).

**Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:**  
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution.  
(b) The product is altered;  
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or  
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

**THE REMEDIATION OF THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT, ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIMITED TO THIS LIMITED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.**

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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Aspen Medical Products, LLC non avrà alcun obbligo ai sensi della presente garanzia limitata nei seguenti casi: