ASPEN SIERRA™ TLSO ADJUSTABLE

THORACIC LUMBAR SACRAL ORTHOSIS
LUMBAL-SAKRAL-THORAKALE ORTHESE

ÓRTESIS TORÁCICA LUMBOSACRA
ORTOSI TORACO-LOMBO-SACRALE

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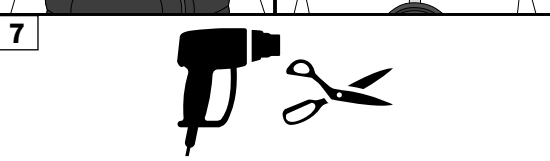
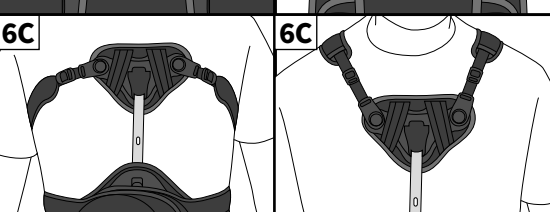
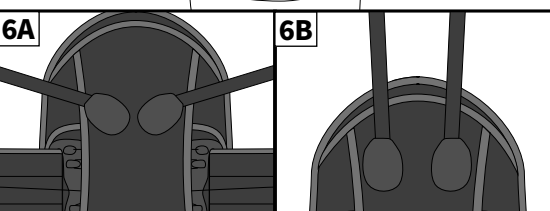
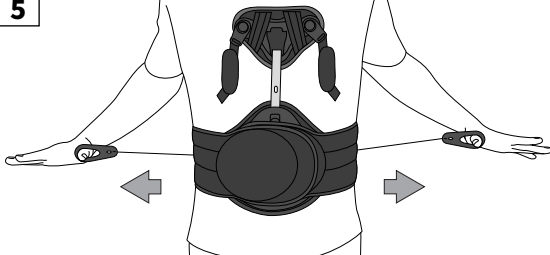
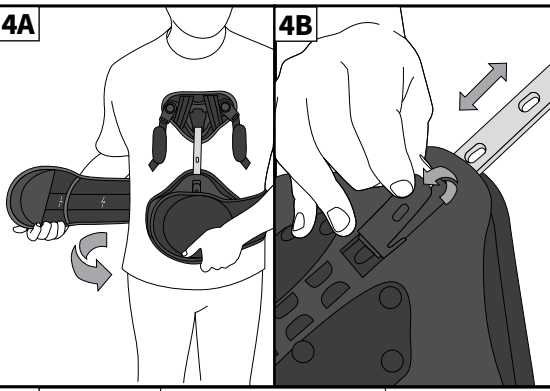
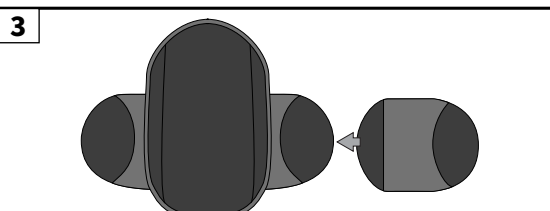
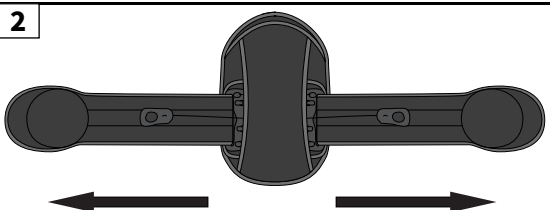
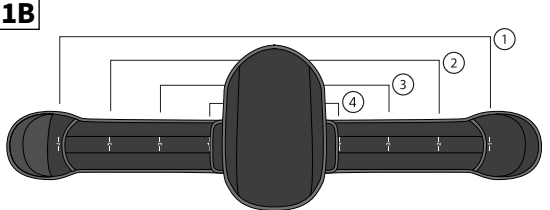
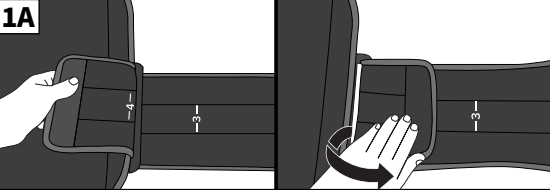
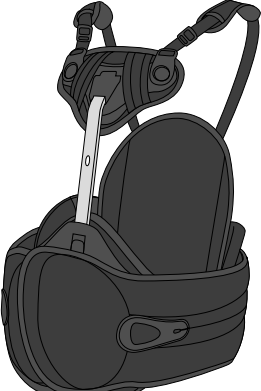
ENGLISH

1. SIZE ADJUSTMENT

A. To adjust the size of the Aspen Sierra TLSO, peel up the inside end of the side panel on each side. Adjust to achieve a proper fit and reattach.
B. Size ranges are only guidelines. Adjust as needed to achieve a snug fit without too much overlap.

Size	Circumference
1	28 – 37 in 71 – 94 cm
2	36 – 43 in 91 – 109 cm
3	42 – 48 in 107 – 122 cm
4	47 – 54 in 119 – 137 cm

2. PREPARATION

Place the pull tabs 2 inches (5 cm) from the plastic in the back of the Aspen Sierra TLSO brace. Pull the ends of the brace away from each other ensuring that it is fully extended.

3. LATERAL PANEL SIZING

Add or remove panels for appropriate sizing.

4. PLACEMENT & ADJUSTMENT

A. Center the back panel as low as possible. Slide abdominal panel and sternal pad sideways as needed to ensure brace and panel are at patient's midline.
B. To determine sternal pad height, pull button back and slide aluminum strut so that the button locks into the appropriate hole.

5. TIGHTENING

Slip your thumbs through the holes in the pull tabs. Pull away from your body. The pull tab controls low compression and the right pull tab controls the upper compression. Adjust the compression as needed. When tightened, secure the pull tabs to the loop in front of the brace.

6. SHOULDER STRAPS

A. Attach control straps to back panel. Straps can be positioned to fit under the arms or over the shoulders.
B. Control straps and sternal pad should fit securely to patient's anatomy. Pull straps to tighten as needed.
C. Torsio control strap setting.
D. Over the shoulder setting.

7. MODIFICATION OPTIONS

This product may be substantially modified by an individual with expertise. For more information, please visit www.aspenmp.com/education/product-modifications.

INTENDED USE

To provide motion restriction and stability of the Thoracic-lumbar-sacral spine.

INDICATIONS

Postoperative support, Degenerative disc disease, Bulging or herniated disc, Fracture management, Spondylolysis, Spondylolisthesis, Spinal stenosis, Kyphosis, Spondylolysis (Osteoarthritis/Degenerative Joint Disease).

CONTRAINDICATIONS

Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

PRECAUTIONS

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

CARE INSTRUCTIONS

Hand wash only at 30° C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

MATERIALS

Nylon, Polyester, HDPE, PVC, Polyurethane, UHMWPE, Aluminium.

NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL

Product may be safely disposed in accordance with local laws.

REPORTING

Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

LIMITED WARRANTY

Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or parts).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.
THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.
This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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PRECAUTIONS

CAUTIONS MUST BE OBSERVED BEFORE USING THE PRODUCT. Use this device only as directed. Do not use it for anything other than its intended purpose. This product is not intended to be used for anything other than its intended purpose. Read the instructions carefully. Do not use it for anything other than its intended purpose. Do not use it for anything other than its intended purpose. Do not use it for anything other than its intended purpose.

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