
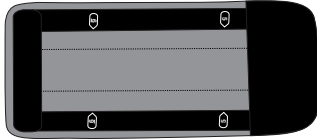
LATERAL EXTENSION PANELS

LATERALE VERLÄNGERUNGSTEILE
PANELES DE EXTENSIÓN LATERAL
PANNELLIDI ESTENSIONE LATERALI
PANNEAUX LATÉRAUX D'EXTENSION
ZIJDELINGSE VERLENGPANELEN
BOČNÍ ROZŠÍŘUJÍCÍ PANELE
LATERALE UDVIDELSESPANELER
ΠΑΛΕΥΡΙΚΑ ΦΥΛΛΑ ΠΡΟΕΚΤΑΣΗΣ
LATERALE FORLENGELSESPANELER
BOCZNE PANELE PRZEDŁUŻAJĄCE
SIVULAAJENNUSPANEELIT
SIDOFÖRLÄNGNINGSPANELER
БОКОВЫЕ УДЛИНИТЕЛЬНЫЕ ПАНЕЛИ
PAINÉIS DE EXTENSÃO LATERAL
橫向延伸护板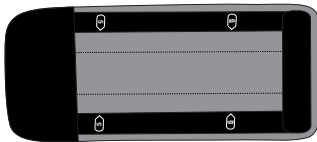

 MEDICAL DEVICE

 SINGLE PATIENT - MULTIPLE USE

 CAUTION







 EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

 Aspen Medical Products, LLC
6481 Oak Canyon, Irvine, CA 92618-5202 USA
aspenmp.com • +1-800-295-2776 • +1-949-681-0200

©2021 Aspen Medical Products, LLC. All Rights Reserved.

Pat. aspenmp.com/patents

LB0027B revised 04/2021

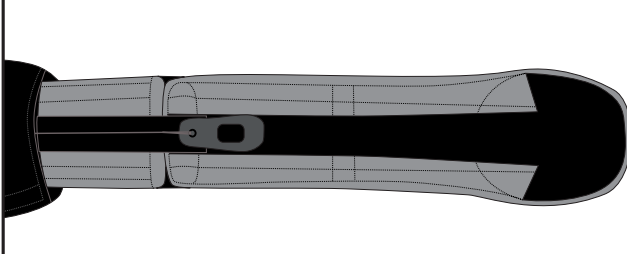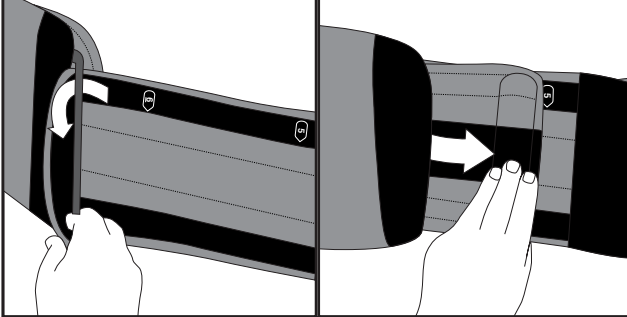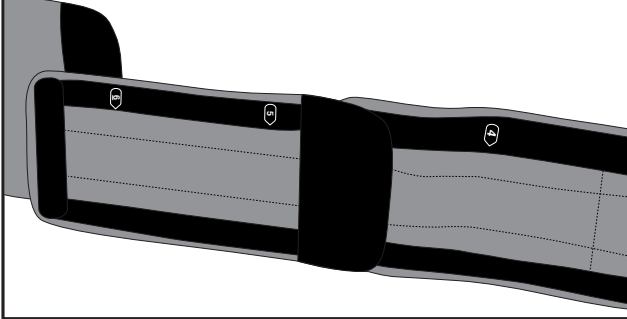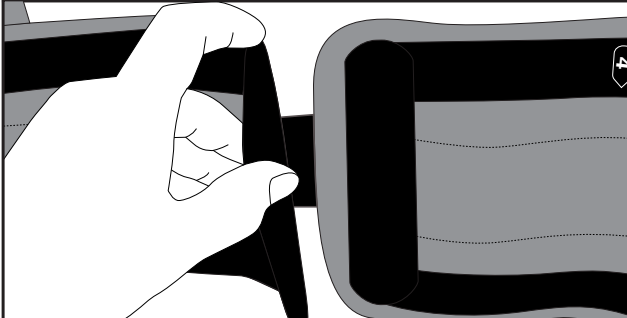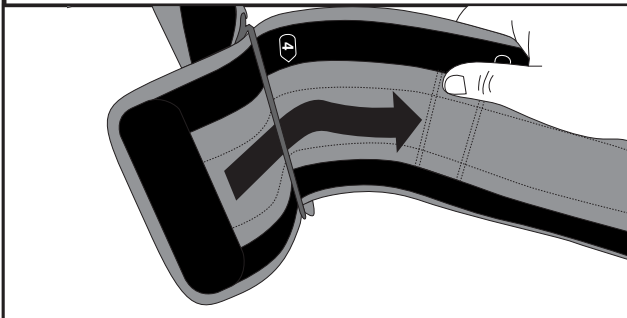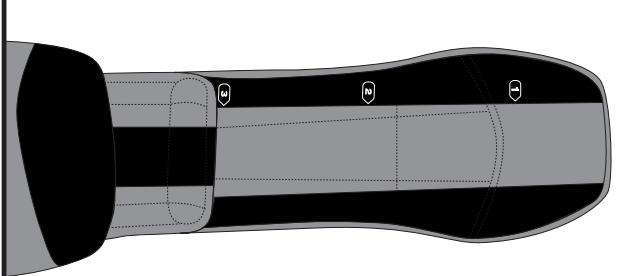ENGLISH

SIZING GUIDE			
Size		Circumference	
3	50-58 in	127-147 cm	
4	56-62 in	142-157 cm	
5	60-68 in	152-172 cm	
6	66-70 in	167-177 cm	

INTENDED USE

Not applicable.

INDICATIONS

Not applicable.

CONTRAINDICATIONS

Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

PRECAUTIONS

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

CARE INSTRUCTIONS

Hand wash only at 30° C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

MATERIALS

Nylon, Polyester, HDPE.

NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL

Product may be safely disposed in accordance with local laws.

REPORTING

Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

LIMITED WARRANTY

Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:

- (a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
- (b) The product is altered;
- (c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
- (d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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(a) Produkt nie został zakupiony w firmie Aspen Medical Products, LLC ani poprzez autoryzowane kanały dystrybucji;