ANKLE FOOT ORTHOSIS KNÖCHEL-FUSS-ORTHESE

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
1A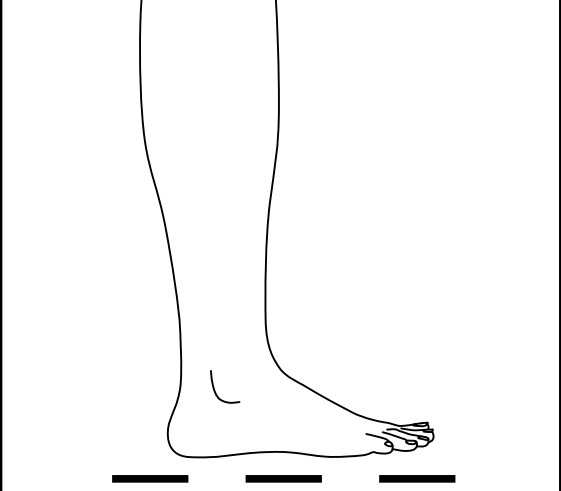

ENGLISH

1. FITTING THE BRACE A. To identify the appropriate AFO size, measure the length of the foot and choose the orthosis based on the measurement.

SIZING CHART table with columns for Posterior AFO, Length, Height, Anterior AFO, and sub-columns S, M, L, XL.

2. ADJUSTMENT A. Mark the footplate to the appropriate size. If removable, use the insole from the shoe to be fitted and trace.

3. APPLICATION A. Remove the insole and apply from the shoe of the side to be fitted. Insert the orthosis and replace the insole.

NOTE: Ensure the orthosis is not in contact with any bony prominences and is comfortable to the patient. If needed, adjust orthosis as appropriate.

To remove, loosen the straps and remove the foot from the shoe. Ensure the orthosis remains in the shoe for easy application.

INTENDED USE An ankle, foot and/or lower leg orthosis for stabilization and positioning.

INDICATIONS Foot drop secondary to neurologic conditions such as stroke, multiple sclerosis, trauma, injury or other conditions that result in weak ankle dorsiflexion function.

CONTRAINDICATIONS Hypersensitivity or allergies to any of the materials from which the brace is made. Neurologic conditions or injury resulting in diminished sensation. Excessive swelling / edema. Moderate to high tone, deformity of foot/ankle or other condition where a brace is not allowed according to medical instructions.

PRECAUTIONS READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

CARE INSTRUCTIONS Clean the soft goods with mild soap and water, rinse thoroughly and allow to air dry. The frame should be wiped down with a damp cloth and allowed to air dry.

MATERIALS Elastic Polyurethane, polyamide, carbon fiber, epoxy resin, nylon

NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

REPORTING Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).


LIMITED WARRANTY Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES.


This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.


2A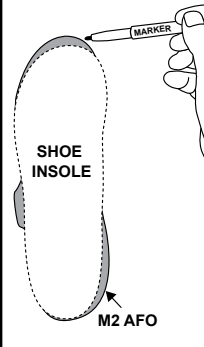

2B

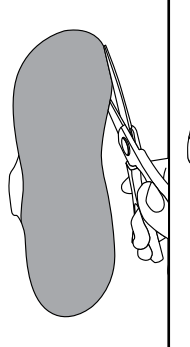
2C

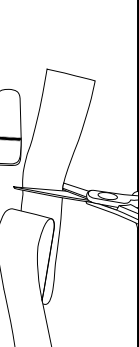3A

3B

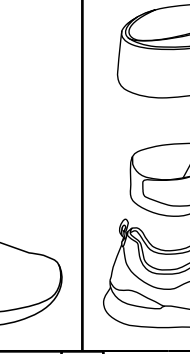3C


3B


3B

3B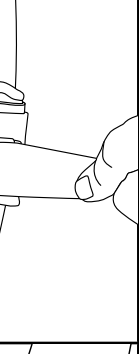



3B

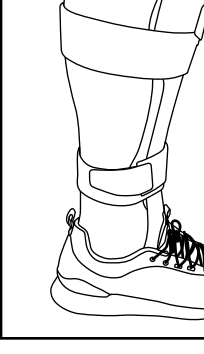3B

3B

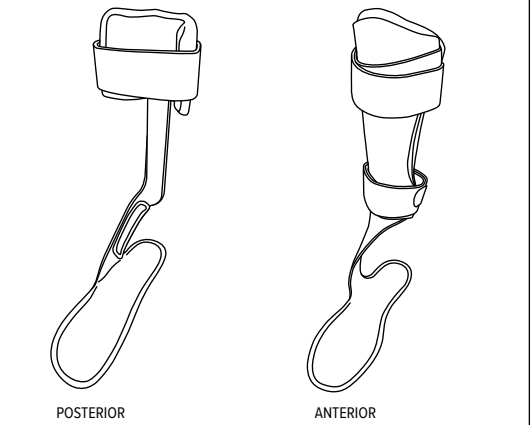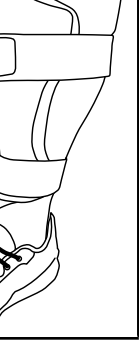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MD MEDICAL DEVICE

Single Patient - Multiple Use

CAUTION

Icons for disposal and recycling instructions

EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands EC REP

Aspen Medical Products, LLC 6481 Oak Canyon, Irvine, CA 92618-5202 USA aspemp.com • +1-800-295-2776 • +1-949-681-0200

©2023 Aspen Medical Products, LLC. All Rights Reserved. LB0067D revised 03/2023

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