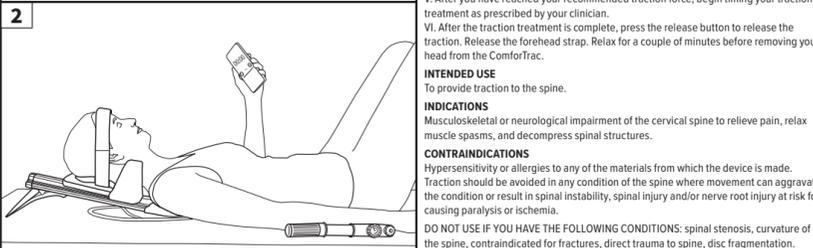
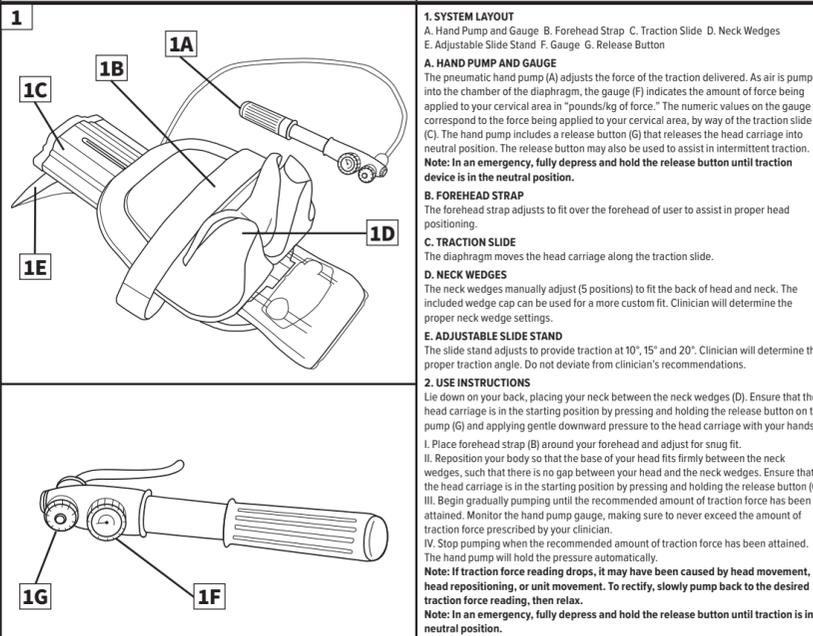
# COMFORTRAC CERVICAL™

CERVICAL TRACTION DEVICE  
VORRICHTUNG ZUR HALSTRACKION  
DISPOSITIVO DE TRACCIÓN CERVICAL  
DISPOSITIVO DI TRAZIONE CERVICALE  
DISPOSITIF DE TRACTION CERVICALE  
CERVICALA TRACTIE-APPARAAT  
KRCŇNÍ TRAKČNÍ ZAŘÍZENÍ  
CERVICAL TRAKTION ANORDNING  
ΣΥΣΤΗΜΑ ΑΥΧΗΚΗΣ ΕΛΞΗΣ  
CERVICAL TREKKANORDNING  
URZĄDZENIE DO TRAKCJI ODCINKA SZYJNEGO  
KAULAN ALUEEN VETOLAITE  
DRAGENHET FÖR HALSKOTOR  
УСТРОЙСТВО ДЛЯ ВЫТЯЖЕНИЯ ШЕЙНОГО ОТДЕЛА  
DISPOSITIVO DE TRACÇÃO CERVICAL  
頸部牽引裝置

**1. SYSTEM LAYOUT**  
A. Hand Pump and Gauge B. Forehead Strap C. Traction Slide D. Neck Wedges E. Adjustable Slide Stand F. Gauge G. Release Button

**A. HAND PUMP AND GAUGE**  
The pneumatic hand pump (A) adjusts the force of the traction delivered. As air is pumped into the chamber of the diaphragm, the gauge (F) indicates the amount of force being applied to your cervical area in "pounds/kg of force." The numeric values on the gauge correspond to the force being applied to your cervical area. By way of the traction slide (C), the hand pump includes a release button (G) that releases the head carriage into neutral position. The release button may also be used to assist in intermittent traction.

**Note: In an emergency, fully depress and hold the release button until traction device is in the neutral position.**

**B. FOREHEAD STRAP**  
The forehead strap adjusts to fit over the forehead of user to assist in proper head positioning.

**C. TRACTION SLIDE**  
The diaphragm moves the head carriage along the traction slide.

**D. NECK WEDGES**  
The neck wedges manually adjust (5 positions) to fit the back of head and neck. The included wedge cap can be used for a more custom fit. Clinician will determine the proper neck wedge settings.

**E. ADJUSTABLE SLIDE STAND**  
The slide stand adjusts to provide traction at 10", 15" and 20". Clinician will determine the proper traction angle. Do not deviate from clinician's recommendations.

**2. USE INSTRUCTIONS**  
Lie down on your back, placing your neck between the neck wedges (D). Ensure that the head carriage is in the starting position by pressing and holding the release button on the pump (G) and applying gentle downward pressure to the head carriage with your hands.

I. Place forehead strap (B) around your forehead and adjust for snug fit.  
II. Reposition your body so that the base of your head fits firmly between the neck wedges, such that there is no gap between your head and the neck wedges. Ensure that the head carriage is in the starting position by pressing and holding the release button (G).  
III. Begin gradually pumping until the recommended amount of traction force has been attained. Monitor the hand pump gauge, making sure to never exceed the amount of traction force prescribed by your clinician.  
IV. Stop pumping when the recommended amount of traction force has been attained. The hand pump will hold the pressure automatically.

**Note: If traction force reading drops, it may have been caused by head movement, head repositioning, or unit movement. To rectify, slowly pump back to the desired traction force reading, then relax.**

**Note: In an emergency, fully depress and hold the release button until traction is in neutral position.**

V. After you have reached your recommended traction force, begin timing your traction treatment as prescribed by your clinician.  
VI. After the traction treatment is complete, press the release button to release the traction. Release the forehead strap. Relax for a couple of minutes before removing your head from the Comfortrac.

**INTENDED USE**  
To provide traction to the spine.

**INDICATIONS**  
Musculoskeletal or neurological impairment of the cervical spine to relieve pain, relax muscle spasms, and decompress spinal structures.

**CONTRAINDICATIONS**  
Hypersensitivity or allergies to any of the materials from which the device is made. Traction should be avoided in any condition of the spine where movement can aggravate the condition or result in spinal instability, spinal injury and/or nerve root injury at risk for causing paralysis or ischemia.

**DO NOT USE IF YOU HAVE THE FOLLOWING CONDITIONS:** spinal stenosis, curvature of the spine, contraindicated for fractures, direct trauma to spine, disc fragmentation.

**PRECAUTIONS**  
Carefully read and familiarize yourself with these directions. Pay particular attention to the following items:

- Recommended traction force (remember, traction should never cause pain).
- Recommended traction duration for each treatment.
- Recommended hold and relax times for intermittent traction applications (if prescribed). Prior to use, remember to have a watch readily accessible to time your treatment.
- Recommended number of traction treatments daily or weekly.
- Recommended neck wedge positioning (Settings 1-5).
- Device should be returned to ambient temperature if stored over 100°F or below 50°F.

**Note: Please read the information and precautions before using the Comfortrac Cervical device. Always consult with your physician or therapist if you have specific questions regarding the use of the device.**

**CARE INSTRUCTIONS**  
Wash with mild detergent. Do not use bleach or other chemicals.

**MATERIALS**  
Aluminum, Nylon, Teflon, Brass, Delrin, Polyester, Steel, Stainless Steel, Copper, HDPE, Silicone, Polyurethane, EVA, ABS, Rubber.

**DISPOSAL**  
Product may be safely disposed in accordance with local laws.

**REPORTING**  
Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

**LIMITED WARRANTY**  
Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or parts).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:  
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;  
(b) The product is altered;  
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or  
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE, OR DEFECT IN THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY OF MERCHANTABILITY, FITNESS FOR PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

**EMERGO EUROPE**  
Westervoortweg 60,  
6827 AT Arnhem  
The Netherlands

**MD MEDICAL DEVICE**

**SINGLE PATIENT- MULTIPLE USE**

**CAUTION**

**EC REP**

Aspen Medical Products, LLC  
6481 Oak Canyon, Irvine, CA 92618-5202 USA  
aspempp.com • +1-800-295-2776 • +1-949-681-0200  
©2023 Aspen Medical Products, LLC. All Rights Reserved.  
LB0078C revised 10/2023

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Aspen Medical Products, LLC no tendrá obligación alguna respecto de esta garantía limitada en los siguientes casos:
(c) Cualquier requisito no suministrado por Aspen Medical Products, LLC fue incluido en el producto.
(d) El producto no se utilizó de acuerdo con las Instrucciones de Uso de Aspen Medical Products, LLC.