### VRTX TLSO

#### THORACIC LUMBAR SACRAL ORTHESIS

#### LUMBAL-SAKRAL-THORAKALE ORTHOSE

#### ÓRTHESIS TORÁCICA LUMBOSACRA

#### ORTESI TORÁCICA SACRO-LOMBARE

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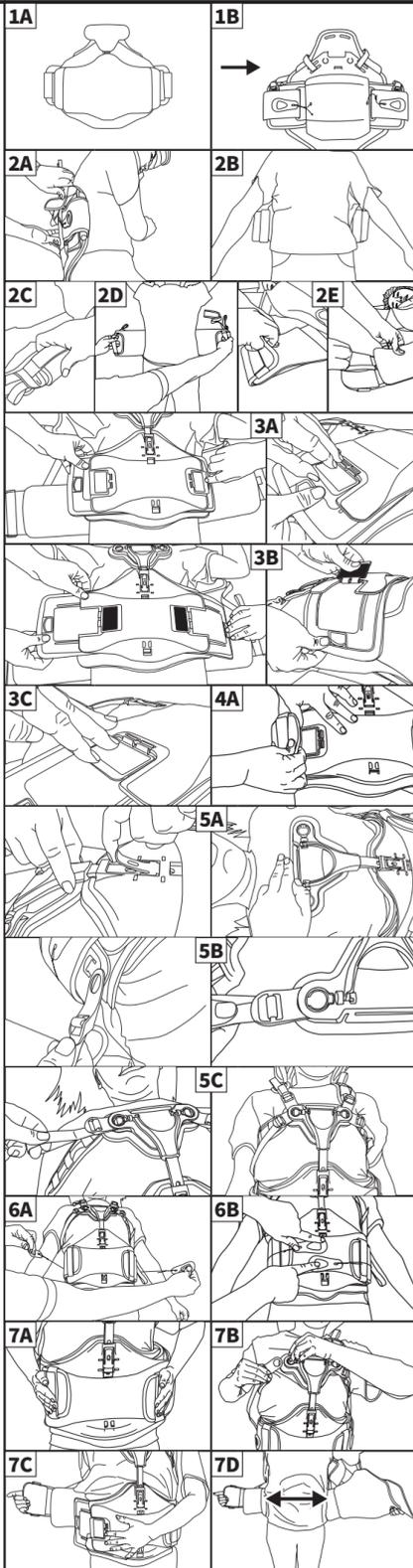
**2. POSITIONING THE POSTERIOR PANEL**  
A. Maintain neutral alignment and with proper support, log roll patient onto their side. Center posterior assembly on patient's back.  
B. Log roll patient onto posterior panel.  
C. Place the handles on each side of the posterior panel.  
D. Pull out the posterior lateral panel from each side ensuring panel remains centered on the patient.  
E. On both sides, remove hook from padding in preparation of anterior panel

**3. POSITIONING ANTERIOR PANEL**  
A. Center the anterior panel on patient. Ensure the securement latch on both sides are lifted off the anterior lateral panels and the panels are free to move.  
B. Pull the anterior lateral panels out so that each panel extends to lateral midline of the patient.  
C. Once the anterior lateral panels are sized appropriately, press the securement latch down onto the panels to lock in place.

**4. ATTACHING ANTERIOR PANEL TO POSTERIOR PANEL**  
A. Attach the posterior panel to the anterior panel one side at a time ensuring the anterior panel remains centered on the patient.

**5. ADJUSTING STERNAL PAD**  
A. Extend the sternal pad by lifting the locking mechanism and sliding the pad to the appropriate height and lock in place. The top of the pad should be about 2 fingers from the sternal notch.  
B. Locate the strap, remove the clip from the lateral panel and clip to the sternal pad. Repeat on the opposite side.  
C. Tighten each side by pulling down on the strap. If desired, accessory straps can be used for both under the arm and over the shoulder attachments.

**6. TIGHTENING THE BRACE**  
A. Pull out on the pull tabs to achieve a snug and comfortable fit.  
B. Secure the pull tabs to the loop material on the front of the brace.

**7. DONNING AND DOFFING**  
A. Place the pull tabs on the side of the brace.  
B. Unclip one of the straps from the sternal pad.  
C. On the same side, remove the belt hook from the anterior panel.  
D. Before donning, pull out the posterior lateral panels to maximize compression.  
E. Reverse the steps to don the brace.

**INTENDED USE**  
To provide motion restriction and stability of the Thoracic-lumbar-sacral spine.

**CLINICAL BENEFIT**  
To address the symptoms of the stated indications through motion restriction and support of the affected muscles.

**INDICATIONS**  
Postoperative support. Degenerative disc disease, Bulging or herniated disc, Fracture management, Spondylyolysis, Spondyloisthesis, Spinal stenosis, Kyphosis, Spondylosis (Osteoarthritis/Degenerative Joint Disease).

**CONTRAINDICATIONS**  
Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

**PRECAUTIONS**  
READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

**CARE INSTRUCTIONS**  
Hand wash only at 30° C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

**MATERIALS**  
Nylon, Polycarbonate, ABS, HDPE, PVC, LDPE, EVA, Acetal, Aluminium, Polyester.

**NOT MADE WITH NATURAL RUBBER LATEX.**

**DISPOSAL**  
Product may be safely disposed in accordance with local laws.

**REPORTING**  
Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

**LIMITED WARRANTY**  
Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or parts).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:  
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;  
(b) The product is altered;  
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or  
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE, IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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