## FLX TLSO 456

### THORACIC LUMBAR SACRAL ORTHOSIS

### LUMBAL-SAKRAL-THORAKALE ORTHOSE

### ORTESIS TORÁCICA LUMBOSACRA

### ORTESI TORACICA SACRO-LOMBARE

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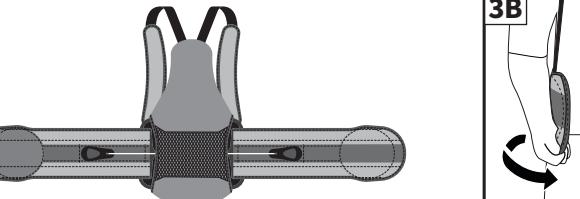
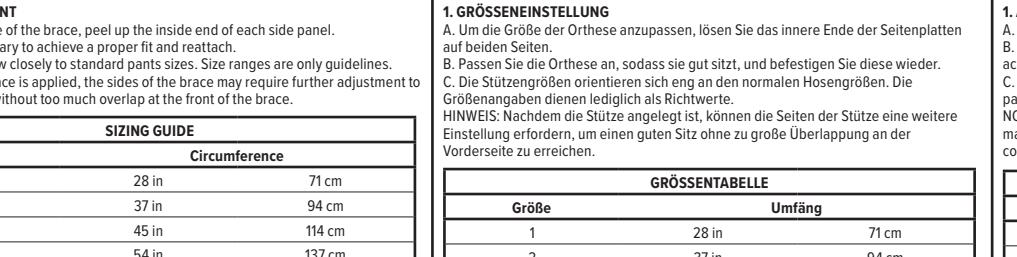
EMERGO EUROPE  
Westervoortsedijk 60,  
6827 AT Arnhem  
The Netherlands

Pat. aspenmp.com/patents

Aspen Medical Products, LLC  
6481 Oak Canyon, Irvine, CA 92618-5202 USA  
aspenmp.com • 1-800-295-2776 • 1-949-681-0200  
©2024 Aspen Medical Products, LLC. All Rights Reserved.  
LB0101A revised 02/2024

### ENGLISH

### SIZING GUIDE

Size	Circumference
1	28 in
2	37 in
3	45 in
4	54 in
	71 cm
	94 cm
	114 cm
	137 cm

### 2. PREPARATION

Place the pull tabs 2 inches (5 cm) from the plastic in the back of the brace. Pull the ends of the brace away from each other ensuring that the brace is fully extended.

### 3. APPLICATION

A. Apply the fully extended brace making sure the back panel is centered and low on the back.

B. Apply the brace tightly, with the right side overlapping the left. When applied properly, both the upper and lower edge of the brace should be snug. If they are not, reapply properly, angle the ends to insure a snug fit at both the upper and lower edges.

C. When tightened, secure the pull tabs to the loop in front of the brace.

### 4. TIGHTENING

A. Slip your thumbs through the holes in the pull tabs. Pull away from your body.

B. The left pull tab controls lower compression and the right pull tab controls the upper compression. Adjust the compression as needed.

C. When tightened, secure the pull tabs to the loop in front of the brace.

### 5. ADJUSTMENT

A. Adjust the upper and/or lower compression by loosening or tightening the pull tabs as needed.

B. When patient is seated, slightly loosening the upper portion may be beneficial.

### 6. STRAP ADJUSTMENT

A. Properly fit shoulder straps.

B. Pull and tighten the shoulder straps.

### 7. MODIFICATION OPTIONS

This product may be substantially modified by an individual with expertise. For more information, please visit [www.aspenmp.com/education/product-modifications](http://www.aspenmp.com/education/product-modifications).

### INTENDED USE

To provide motion restriction and stability of the Thoracic-lumbar-sacral spine

### INDICATIONS

Postoperative support, Degenerative disc disease, Bulging or herniated disc, Fracture management, Spondylolisthesis, Spondylolisthesis, Spinal stenosis, Kyphosis, Spondylolisthesis (Osteoarthritis/Degenerative Joint Disease)

### CONTRAINdications

Hypersensitivity or allergies to any of the materials from which the brace is made.

Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

### PRECAUTIONS

RECOMMENDATIONS BEFORE USE: Proper fitting of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

### CARE INSTRUCTIONS

Hand wash only at 30°C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

### MATERIALS

Nylon, Polyester, HDPE, PVC, Polyurethane, UHMWPE.

### NOT MADE WITH NATURAL RUBBER LATEX

DISPOSAL

Product may be safely disposed in accordance with local laws.

### REPORTING

Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

### LIMITED WARRANTY

Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or part.

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:

(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;

(b) The product is altered;

(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product;

(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES ARISING OUT OF THE USE OF THE PRODUCT. THE FOREGOING ALSO APPLIES TO THE PARTS OF THE PRODUCT OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS. THESE PARTS ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC

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