

LOWER SPINE PAD WASHING INSTRUCTIONS

WASCHANWEISUNGEN FÜR UNTERES POLSTER DER WIRBELSÄULE
INSTRUCCIONES DE LAVADO DE LA ALMOHADILLA DE LA COLUMNA INFERIOR
ISTRUZIONI PER IL LAVAGGIO DELL'IMBOTTITURA POSTERIORE INFERIORE
INSTRUCTIONS DE LAVAGE DES COUSSINS LOMBAIRES
WASVOORSCHRIFTEN PAD ONDERSTE WERVELKOLOM
POKYNY K MYTÍ PODŁOŽKY DOLNÍ ČÁSTI PĚTĚŘE
INSTRUKTIONER FOR RENGØRING AF NEDRE RYGSLØJLE PUDE
ΟΔΗΓΙΕΣ ΠΛΥΣΙΜΑΤΟΣ ΚΑΤΩ ΜΑΞΙΛΑΡΙΟΥ ΣΠΟΝΔΥΛΙΚΗΣ ΣΤΗΛΗΣ
VASKEINSTRUKSJONER FØR NEDRE RYGGGRADPUTE
INSTRUKCJE CZYSZCZENIA PODKŁADKI POD DOLNY ODCINEK KRĘGOSŁUPA
ALEMMAN SELKÄRANKAPEHMUSTEEN PESUOHJEET
TVÄTTINSTRUKTIONER FÖR NEDRE RYGGGRADSDYNAN
ИНСТРУКЦИИ ПО ОЧИСТКЕ НИЖНЕЙ НАКЛАДКИ ДЛЯ ПОЗВОНОЧНИКА
INSTRUÇÕES DE LAVAGEM DA COLUNA INFERIOR
下部脊垫清洗说明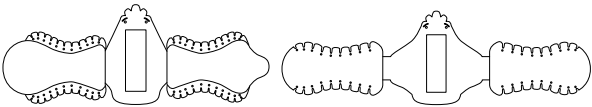


ASPEN, SHORT/TALL LSO

ASPEN, LSO LOW PROFILE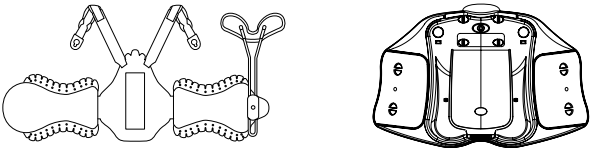

ASPEN, SHORT/TALL TLSO

CONTOUR LSO

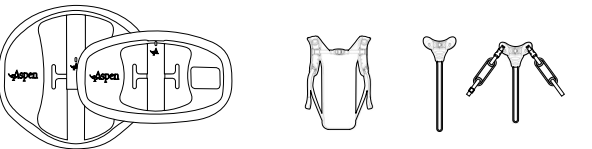RIGID ANTERIOR PANEL (RAP)

SHOULDER STRAP

CHEST PANEL

SPK

(CONTOUR TLSO)

MD MEDICAL DEVICE

1P SINGLE PATIENT- MULTIPLE USE

CAUTION




CE **EC** **REP** **EMERGO EUROPE**
Westervoortsedijk 60,
6827 AT Arnhem
The Netherlands

Aspen Medical Products, LLC
6481 Oak Canyon, Irvine, CA 92618-5202 USA
aspenmp.com • +1-800-295-2776 • +1-949-681-0200

©2023 Aspen Medical Products, LLC. All Rights Reserved.

Pat. aspenmp.com/patents

LS1800D revised 03/2023

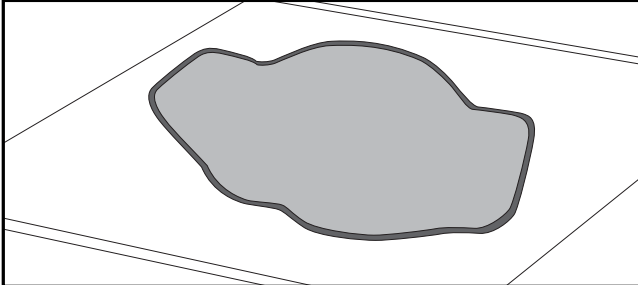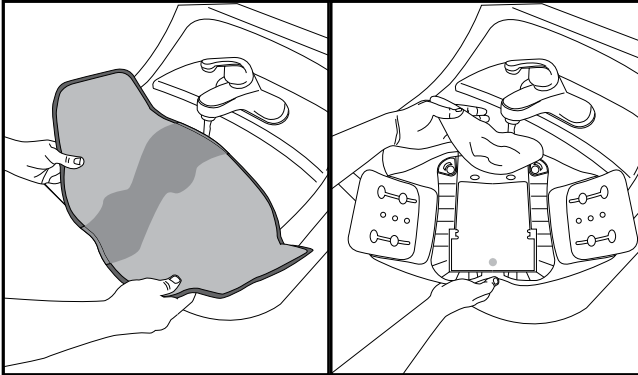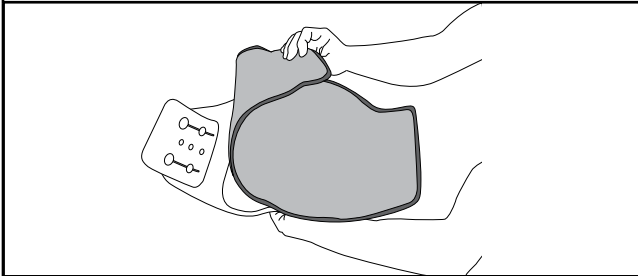ENGLISH

1. PAD REMOVAL

Remove the pads from the product by peeling the pad away from the hook dots.

2. CLEANING

A. Hand wash pads with mild soap and warm water. Do not machine wash or use bleach. Rinse thoroughly.

B. The plastic shell of the product may be wiped with mild soap and water. Rinse and towel dry before re-attaching pads.

3. DRYING

Squeeze out excess water by pressing pads with a towel. Allow to completely air dry by laying on a towel. Do not use heat to dry. Use replacement pads during drying time.

INTENDED USE

Not applicable.

INDICATIONS

Not applicable.

CONTRAINDICATIONS

Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

PRECAUTIONS

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Permanent injury may result from an improper use of this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the accompanying limited warranty for further information.

CARE INSTRUCTIONS

Hand wash only at 30° C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

MATERIALS

Nylon, Cotton, Polyurethane.

NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL

Product may be safely disposed in accordance with local laws.

REPORTING

Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

LIMITED WARRANTY

Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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